

## Cleaning and Disinfection of Unisensor UniTip Catheters

Besides the recommended cleaning and disinfection solutions in the UniTip instructions for use, UniTip Catheters may be cleaned and disinfected with the listed solutions/systems **without damaging the UniTip Catheters (material compatibility and performance)**.

**Important:** The recommended immersion time of the authoritative instruction for use must be maintained.

### Cleaning Solutions:

Enzol®	Johnson&Johnson
Deconex 36 intensiv®	Borer Chemie

### Cleaning and Disinfection Solutions (particularly tested and recommended for UniTip High Resolution Catheters):

Sekusept® Plus 4% Solution/15min immersion time	Ecolab
Sekusept® aktiv 2% Solution/15min immersion time	Ecolab
Tristel TRIO Sporidical Wipes System	Tristel Solution Limited
Tristel Fuse for Instruments (placed in Stella5 Disinfection System)	Tristel Solution Limited
Resert™ HLD High Level Disinfectant	STERIS Corporation

### Disinfection Solutions:

Cidex OPA®	Johnson&Johnson
ReleyOn™PeraSafe™	DuPont
Deconex 53 Plus®	Borer Chemie

### Important:

The USER is responsible for cleaning / disinfection / sterilization of the UniTip catheter. Take care that only proven procedures and solutions are used for cleaning / disinfection / sterilization, and that the parameters always comply with the validation.

Additionally, please pay attention to the laws / regulations / guidelines of your country as well as the hygienic instructions of your hospital.